

## FR2 AED MAINTENANCE CHECKLIST

|  |  |  |  |  |  |  |  |  |
|--|--|--|--|--|--|--|--|--|
| Date   |  |  |  |  |  |  |  |  |
| AED Unit<br>Clean, no dirt or contamination; no damage present   |  |  |  |  |  |  |  |  |
| Supplies Available:<br>a. 2 sets of Adult defibrillation pads, within expiration date, undamaged<br>b. 1 set of Pediatric defibrillation pads within expiration date, undamaged<br>c. Ancillary supplies:<br><ul style="list-style-type: none"> <li>• Towel</li> <li>• Scissors</li> <li>• Razor</li> <li>• CPR mask</li> <li>• Gloves</li> </ul> d. Store unopened battery within install before date |  |  |  |  |  |  |  |  |
| Status Indicator:  |  |  |  |  |  |  |  |  |
| Inspected by:  |  |  |  |  |  |  |  |  |
| Remarks, Problems, Corrective Actions:   |  |  |  |  |  |  |  |  |

Prepared by Emergency Skills, Inc.  
 Daily, check AED status indicator.  
 Monthly, perform thorough check, Date and initial upon completion.  
 Please make copies of this form as needed. Store completed forms in a safe place.  
 To order supplies, call the Dept of Education - Sherry Fusco (718-391-8382), Celeste McGee (718-391-8566) or  
 Donovan Lauther at (718) 391-8227.